








ALVERNO LABORATORIES

SPECIMEN COLLECTION GUIDE MOLECULAR

Specimen collection instructions and guidelines may change. We recommend checking the collection manual for the most up-to-date collection information at www.AlvernoLabs.com

Reviewed May 2022

Reach us at 800-937-5521

<p><u>THIN PREP PAP</u></p>  <p>ORDER# 144598 ⇒ HPV ⇒ GC/Chlamydia</p>	<p><u>ABBOTT MULTI-COLLECT SPECIMEN COLLECTION KIT</u></p>  <p>ORDER# 176526 Intended for the collection and transportation of male and female, swab and urine specimens for the detection of Chlamydia trachomatis and Neisseria gonorrhoeae</p>	
<p><u>BBL CULTURE SWAB LIQUID STUART</u></p> <p>ORDER# 193216</p> <p><u>OR E-SWAB</u></p> <p>ORDER# 132433</p> <p>⇒ MRSA or MSSA by PCR</p>  	<p><u>BD AFFIRM VAGINITIS PANEL</u></p> <p>ORDER# 108401</p> <p>⇒ Gardnerella ⇒ Trichomonas ⇒ Candida sp.*</p> <p>*does NOT identify the species of Candida yeast</p> 	<p><u>BD Max Vaginitis Panel</u></p> <p>ORDER# 181279</p> <p>⇒ Bacterial Vaginitis ⇒ Trichomonas ⇒ Candida species will be identified</p> 
<p><u>E-SWAB</u></p> <p>ORDER# 132433</p> <p>⇒ Group B Strep by PCR ⇒ Vaginal/Rectal E-SWAB</p> 	<p><u>STERILE SCREW CAP CUP (no additive)</u></p> <p>ORDER# 153706</p> <p>⇒ Respiratory Wash/Aspirate</p> <ul style="list-style-type: none"> * Respiratory 21 Pathogen Panel * B. pertussis and parapertussis * Mycoplasma pneumoniae <p>⇒ HSV 1 and 2</p> <ul style="list-style-type: none"> * Respiratory Wash / CSF <p>⇒ Varicella Zoster Virus (VZV)</p> <ul style="list-style-type: none"> * CSF <p>⇒ C. diff by PCR</p> <ul style="list-style-type: none"> * Stool <p>⇒ Enteric Pathogen Panels</p> <ul style="list-style-type: none"> * Stool (bacterial and parasitic) <p>⇒ Norovirus</p> <ul style="list-style-type: none"> * Stool <p>⇒ CMV Qualitative by PCR:</p> <ul style="list-style-type: none"> * CSF, Urine, Respiratory Wash <p>⇒ Meningitis/Encephalitis Pathogen Panel</p> <ul style="list-style-type: none"> * CSF 	
<p><u>UTM (Universal Transport Media)</u> ORDER# 123823</p> <p>By PCR:</p> <ul style="list-style-type: none"> ⇒ Varicella zoster virus (VZV) ⇒ Herpes virus 1&2 ⇒ Mycoplasma pneumoniae ⇒ B pertussis and parapertussis ⇒ Respiratory 21 Pathogen Panel  <hr/> <p><u>UTM (Abbott)</u> ORDER# 197838</p> <ul style="list-style-type: none"> ⇒ SARS-CoV-2 ⇒ Flu A / Flu B / RSV virus panel ⇒ Flu A / Flu B virus panel 		
<p><u>LAVENDER TOP TUBE: WHOLE BLOOD</u></p> <p>ORDER# 9603</p> <ul style="list-style-type: none"> ⇒ Factor V Leiden Mutation by Real Time PCR ⇒ Factor II Mutation by Real Time PCR ⇒ CD4/CD8 (T-Cell Subsets) ⇒ Lymphocyte Subsets (T, B, and NK cells) ⇒ Lymphoma/Leukemia Immunophenotyping by FlowCytometry ⇒ Fetomaternal Bleed by FlowCytometry ⇒ HLA B27 Detection by FlowCytometry ⇒ Next Generation Sequencing, Myeloid Neoplasia Panel 		
<p><u>LAVENDER TOP TUBE: PLASMA*</u></p> <p>ORDER# 9603</p> <ul style="list-style-type: none"> ⇒ CMV Qualitative PCR ⇒ CMV Quantitative PCR ⇒ HCV Quantitative by PCR ⇒ HIV Quantitative by PCR** ⇒ HBV Quantitative By PCR <p>** requires plasma from 2 lavender tubes</p> <p><i>*Submit plasma in a transfer tube</i></p> 		
<p><u>UROVYSION</u></p> <p>ORDER# 135884</p> <p>⇒ For bladder cancer test collection use Urocyte Kit</p> 		<p><u>PARA-PAK SAF FIXATIVE</u></p> <p>ORDER# 11939</p> <p>⇒ Enteric Parasite Panel</p> <p><u>PARA-PAK C&S CARY-BLAIR MEDIA</u></p> <p>ORDER# 23205</p> <ul style="list-style-type: none"> ⇒ Gastrointestinal 22 Pathogen Panel ⇒ Enteric Bacterial Panel ⇒ Extended Enteric Bacterial Panel ⇒ Norovirus  
<p><u>GREEN TOP TUBE: SODIUM HEPARIN</u></p> <p>ORDER# 9604</p> <p>⇒ Hematological FISH testing for peripheral blood or bone marrow aspirate</p> 		